FR

PROPRIÉTÉS

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EN

PROPERTIES

The DETARTRINE range is available as ready-to-use pastes designed for dental scaling and polishing.

DETARTRINE and **DETARTRINE Z** are pink pastes, with a nice taste. Their mechanical abrasive action is due to the presence of silica, which makes it possible to complete the removal of dental tartar, after using an ultrasonic or hand scaler, without causing erosion of dental enamel. DETARTRINE Z contains zirconium particles. These particles look like flat scrapers, the sides of which are separated by a very hard edge. When this paste is rubbed against a rough surface, the particle edges shave the bumps off. The essential oils incorporated into DETARTRINE and DETARTRINE Z give a feeling of freshness, which is very much appreciated by the patients. In addition, they have a slight balsamic effect: they relieve the irritation that mechanical scaling might cause on sensitive gums.

INDICATIONS

- Finishing of scaled surfaces,
- Removal of food and tobacco stains,
- Fine polishing of fillings.

INSTRUCTIONS FOR USE

Take the required amount of paste with a spatula and place it onto a polishing wheel.

Note: Mouth rinsing can be performed during or at the end of scaling.

PRECAUTIONS FOR USE

Possible allergic reaction to formaldehyde.

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