fr

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en

INSTRUCTIONS FOR USE

Please read instructions carefully prior to use.

Priorly absorbable collagens for use as hemostatic agent after tooth extractions and oral surgery

APPLICATION FORM

The sterile absorbable SeptoCone is used as hemostatic agents after tooth extractions and oral surgery. The application of SeptoCone into the defect of cavity supports the stabilization of the produced blood clot whilst helping to control bleeding at the application site.

COMPOSITION

1 SeptoCone cone contains 22.4 mg of native collagen fibrils of equine origin.

INTENDED USE

The sterile absorbable SeptoCone is used as hemostatic agents after tooth extractions and oral surgery. SeptoCone can be combined with fibrin glue in patients with an increased risk of bleeding.

MODE OF APPLICATION

- After removal from the sterile packaging, SeptoCone can be cut to the desired shape and size with sterile instruments.
- SeptoCone is used in dry state with light pressure to achieve better adhesion and hemostasis.
- SeptoCone is inserted into the extraction socket or the cavity as required.
- Once inserted into the desired cavity SeptoCone can be sutured over and /or held in place with fibrin glue as required.
- Due to the affinity of collagen with blood-covered surfaces, dry instruments and gloves should be used when administering the SeptoCone.
- Due to the absorbability of the SeptoCone it does not have to be removed. SeptoCone will be absorbed within 3-4 months, absorption time may differ as it is dependent upon multiple patient-specific factors, including but not limited to site of implantation, comorbidities, and blood perfusion.
- For dental professional use only

Medical treatment of severe bleeding disorders is mandatory prior to the beginning of the procedure.

Unless otherwise prescribed, SeptoCone can be applied directly into the extraction alveola. The SeptoCone moulds itself into the site of the extraction or excision wound, stop bleeding, and will be absorbed.

CONTRAINDICATIONS

SeptoCone must not be used:

- on patients with a known hypersensitivity to products of equine origin,
- in infected areas,
- during pregnancy and lactation.

WARNINGS

Chlorine-releasing antiseptic agents, tannins and caustics should not be used in combination with collagen.

SeptoCone is not intended to be sutured through; doing so may compromise the performance of the SeptoCone.

SIDE EFFECTS

Due to the xenogenic origin of the collagen, there is a possible risk of intolerance reaction, despite the confirmed low antigenicity.

FURTHER NOTES

Due to the affinity of collagen to bleeding surfaces, dry instruments and gloves must be used when applying SeptoCone. The product is intended for single use only. SeptoCone must not be reused, there is a risk of infection for the patient and/or user and the functionality of the product for the patient and /or user cannot be guaranteed.

Remaining material, left over due to cutting for example, must be disposed of according to local procedures and guidelines.

NOTES FOR POST-OPERATIVE CARE FOLLOWING

DENTAL SURGERY

To be avoided:

- Rinse your mouth out on the day of surgery (may dislodge clot).
- Drink alcohol or hot drinks on the day of surgery (may increase blood pressure or induce hyperaemia).
- Smoke for at least 24 hours (causes vasoconstriction).
- Carry out physical activity on the day of surgery (increasing blood pressure).

Recommended:

- Sip lukewarm drinks carefully (helps avoid clot removal by washing out socket).

- Bite on gauze or clean, damp handkerchief for 10 minutes if area bleeds, preferably sitting upright in a chair.
- Rinse mouth gently with hot salt mouthwash or Chlorhexidine (0,2 %) mouthwash after 24 hours. Continue regular mouthwash use over the following week, especially after meals.
- Clean teeth as normal, avoiding area of surgery for 24 hours.

STERILITY

SeptoCone may not be resterilised and opened single packs must be discarded. Do not re-use SeptoCone in whole or in part, as it may compromise sterility and /or the performance of SeptoCone.

STORAGE

Store between 0° C -25° C.

Store protected from moisture and direct sunlight.

Do not refrigerate.

Store out of the sight and reach of children.
PRODUCT CONFORMS TO THE ESSENTIAL REQUIREMENTS OF THE COUNCIL DIRECTIVE 93/42/EEC CONCERNING MEDICAL DEVICES.

DESCRIPTION OF THE SYMBOLS USED ON THE PACKAGE

REF	REFERENCE NUMBER
LOT	BATCH NUMBER
	USE BY YEAR – MONTH – DAY
	CONSULT INSTRUCTIONS FOR USE
	DO NOT RE-USE
	DO NOT RESTERILIZE
	DO NOT USE IF PACKAGE IS DAMAGED
STÉRILISÉ	STERILIZED USING ETHYLENE OXIDE
	TEMPERATURE LIMIT
	PROTECT FROM DIRECT SUNLIGHT
	HIBC CODE
	KEEP DRY
	MANUFACTURER
	CE MARKING AND IDENTIFICATION NUMBER OF THE NOTIFIED BODY

For dental professional use only
GA 0089, Status 2021-03
REF : DK1010SD

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