Septofil N

fr - French	2
en - English.....	3
es - Spanish	4
de - German	5
it - Italian	6
pt - Portuguese.....	7
nl - Dutch	8
pl - Polish.....	9
el - Greek.....	10
bg - Bulgarian	11
cs - Czech	12
da - Danish	13
et - Estonian	14
hu - Hungarian.....	15
lt - Lithuanian.....	16
lv - Latvian.....	17
ro - Romanian.....	18
sk - Slovak.....	19
sl - Slovenian.....	20
sv - Swedish.....	21

Septofil N

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EC REP BIOGERI: Am Pfeilshof 12, 22393 Hamburg, Germany

Distributed by :
SEPTODONT
www.septodont.com

Septofil N

Septofil N is a specially knitted cord, made of 100% cotton. This Knitted form has excellent resiliency, elasticity and absorbency which provides great convenience.

Options:

Septofil N is available in 6 diameters identified by different colors:

Septofil N 0 ⁴ (The finest)	black/blue	0.5 mm
Septofil N 0 ³ (Ultra-fine)	black/gray	0.7 mm
Septofil N 0 ² (Extra-fine)	brown/yellow	0.82 mm
Septofil N 0 (Fine)	purple/light-purple	0.99 mm
Septofil N 1 (Medium)	blue/light-blue	1.2 mm
Septofil N 2 (Thick)	green/light-green	1.35 mm

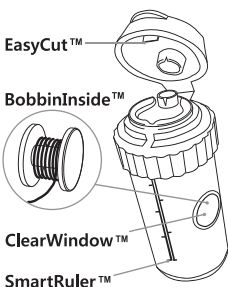
Contents:

Dispenser with 254cm (100 inch) and BobbinInside™

Precautions:

Check the condition of gingival tissue (eg, depth or thickness)

Procedure:

1. Select a cord with an appropriate diameter.
2. Cut the cord to the required length with EasyCut™ and SmartRuler™. If necessary, the cord can be impregnated with a hemostatic solution.
3. Insert the cord into the sulcus by means of a retraction cord packer.
4. Leave the cord about 5 minutes in the sulcus.
5. Remove the cord just before taking the impression. Slightly moisten the cord to make its removal easier and reduce gingival

Caution:

- For professional use only.
- To avoid a possible permanent displacement of the gingiva, the cord should not be remained for longer than 20 minutes in the sulcus.
- After finishing the procedure, make sure to remove the cord from the sulcus.

Do not re-use

Symbols (EN980):

EASTDENT Co., Ltd: #601-1, 64 Gugal-ro, Giheung-gu, Yongin-si, Gyeonggi-do, 16972, Korea

EC REP BIOGERI: Am Pfeilshof 12, 22393 Hamburg, Germany

Distributed by :
SEPTODONT
www.septodont.com

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