**SeptoTape** - Tape for dental isolation  
SeptoTape is indicated for teeth isolation during acid conditioning as well as for the application of adhesives, composites and other reagents. Made of inert and flexible material, the tape fits on the tooth, protecting it against undesirable aggressions during the restorative procedure performed on the adjacent tooth. Width: 18mm, Thickness: 0,07mm. **Use:** Cut one tape segment as required. Introduce the tape in the interproximal area, pull the ends by lingual and vestibular and smoothly press them against the adjacent tooth. Remove the tape after performing the procedure. **Warnings, precautions and restrictions:** Professional use only. Single use. Its reuse can cause cross contamination. When there is proven allergy to any of its components, the product must not be used. Any serious incident related to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established. **Compositon:** Polytetrafluorethylen. **Storage:** To be stored in a cool and dry place, protected from sunlight. **Presentation:** 0111050005 - SeptoTape - Blister with 5m of dental isolation tape

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SEPTODONT, Inc
Obelis S.A.