(EN)

SeptoDiamond Strips

Diamond steel strips for finishing of restorations and orthodontic stripping

SeptoDiamond Strips is a diamond steel strip indicated for finishing of filling materials and orthodontic stripping. Available in fine (yellow) and medium (red) grits.

Use: Insert the SeptoDiamond Strips in the interproximal area with the abrasive side turned to the surface to be worn (stripping or finishing). Conduct longitudinal movements, back and forth. The abrasive side is labeled according to the strip's grit: red (medium) and yellow (fine). To achieve the best results it is recommended to use both strips, starting with the medium grit and concluding with the fine grit.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. The contact of the strip with the gum may cause minor injury to the patient. It is recommended to protect the papilla with a wedge or other protection method.

Sterilization: The product is single use, however the professional can choose to sterilize prior to the use. Before sterilization, proceed with the cleaning using water and enzymatic detergent. Sterilizable in autoclave (126°C, 147kPa, 16min).

Composition: Stainless steel and diamond.

Storage: To be stored in a cool and dry place, protected from sunlight.

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