(EN)

SeptoProphy Strips

Non-abrasive steel strip with circular micro blades for interproximal tartar removal

SeptoProphy Strips is an indispensable aid in prophylaxis procedures, since it easily accesses the interproximal region, where its active face effectively provides the removal of dental calculus. The circular micro-blades, located around the holes in one face of the strip, has as main characteristic the action against calculus, thus preserving the enamel and restorative materials integrity.

Use: Insert the strip in the interproximal space, placing its active face turned to the desired tooth. Hold the strip by the ends, pull it against the tooth and apply back and forth longitudinal movements until achieve the desired effect. Remove the strip.

Warnings, precautions and restrictions: Professional use only. Single use only. Its reuse can cause cross contamination. When there is proven allergy to any of its component, the product must not be used. The strip in contact with the gum can cause injuries to patient, we recommend the protection of the gum. The reuse may affect the product's performance.

Sterilization: The product is single use, however the professional can choose to sterilize prior to the use. Before sterilization, proceed with the cleaning using water and enzymatic detergent. Sterilizable in autoclave (126°C, 147kPa, 16min).

Composition: Stainless steel.

Storage: To be stored in a dry place, fresh and sheltered from sunlight.

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EC REP: Obelis S.A, Bd. Général Wahnis 53, 1030 Brussels, Belgium, Tel: +32 (0) 27 55 55 54, E-mail: mail@obelis.net
Distributed in USA by: SEPTODONT, Inc, Lancaster, PA 17601 - USA