(EN)

SeptoCompo Shape

Silicone brushes for composites shaping

SeptoCompo Shape were developed to promote easiness and precision in shaping composites both direct and indirect restorations. The kit contains a straight instrument, which facilitates the access to the vestibular anterior/posterior teeth, as well as for laboratory work, and an angled instrument, which allows an easier and more comfortable access to the lingual/palatinal region of the anterior/posterior teeth and to the occlusal region of posterior teeth. The brushes are interchangeable, made of silicone rubber, anti-adherent and available in four different shapes and two different hardness (hard and soft tips), which allows to shape the composite to obtain the desired tooth anatomy. The possibility of rotating the brushes on the end of the instrument provides comfort during its use. Using SeptoCompo Shape avoids the typical lines and marks caused by hard instruments and, consequently, the differences of shine and color that they may cause.

Use: Select the instrument and brush (tip), which best fits to the clinical case. Place the brush on the end of the instrument, rotate the brush to reach the best position. Shape the composite with light touches, starting with the hard brushes (blue) and finishing with the soft brushes (white). chamfered edge cut; Single chamfered edge cut; Double angled chamfered edge cut; Flame.

Warnings, precautions and restrictions: Professional use. It there is proven allergy to any of its components, the product must not be used.To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. It is recommended to clean the silicone brushes with alcohol after each use to remove composites excess in order to ensure their anti-adherence. Sterilization methods may cause a slight color changing in the brushes, which does not affect its functionality. Product not recommended for: condensable composites, flow composites, amalgams, glass ionomer cement and self-curing acrylic resins.

Sterilization: Autoclavable (126 °C, 147kPa, 16min). Before sterilization, proceed with the cleaning using water and a specific detergent.

Composition: Instruments: Polysulfone. Soft brushes: Silicone and titanium dioxide. Hard brushes: Silicone, titanium dioxide, aluminum oxide and atomic colorants.

Storage: To be stored in a cool and fresh place, protected from sunlight.

0105020019 – SeptoCompo Shape Soft/Hard – 2 instruments and 8 brushes

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