SeptoCure

Instruments to condense composites

(EN)

SeptoCure is indicated to condense composites resins in clinical procedures, since it allows to light cure through their tips while the composites are being hold under pressure on the bottom of the cavity during polymerization, reducing the risk of marginal leakage. Two instruments with four different tips, developed specifically for the application of this technique.

Use: Select among the four available tips the one which best adapts to the proximal box. Take the first increment of resin to the cavity, keeping it under pressure while being polymerized. Repeat the procedure, layer by layer, as often as necessary.

Warnings, precautions and restrictions: Professional use. If there is proven allergy to any of its components, the product must not be used. The instrument should be sterilized in autoclave before use. Sterilization performed with any method other than autoclave can damage the product.

Sterilization: Autoclavable (126 °C, 147 kPa, 16 min). Before sterilization, proceed with the cleaning using water and enzymatic detergent.

Composition: Polysulfone.

Storage: To be stored in a cool and dry place, protected from sunlight.

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SEPTODONT, Inc
OBELIS S.A.