SeptoMatrix Cervical

Transparent matrix for Class V restorations

SeptoMatrix Cervical is an instrument used for modelling composites in Class V restorations, composed by five different sizes and flexible shell shapes. The matrices provide a perfect subgingival adaptation in the cervical region of the tooth when kept under pressure, besides allowing the visualization and polymerization of the composite due to its transparency. The result is an excellent tooth anatomy and reduced time in the procedure.

Use: Select among the five available sizes the one which better adjusts to the tooth to be restored. Adjust the matrix to the instrument. If the cavity has a subgingival extension, promote the gingival retraction. Insert the last layer of composite, compress the matrix against the tooth forcing the composite to flow to incisal / occlusal region. Remove the excesses from the edges of the matrix and polymerize for 10 seconds through the matrix. Remove the matrix and complete the polymerization according to the recommendations of the manufacturer of the composite. Perform the finishing and polishing procedures.

Available in 5 sizes.

Warnings, precautions and restrictions: Professional use. The matrices are for single use only. Its reuse can cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operating field.

Sterilization: The instruments are autoclavable (126°C, 147kPa, 16min.). Before sterilization, proceed with the cleaning using water and enzymatic detergent.

Composition: Instruments: Polysulfone, Matrices: Polypropylene.

Storage: To be stored in a cool and dry place, protected from sunlight.

Presentation:

0105040016 – SeptoMatrix Cervical – 2 instruments + 10 matrices

0105040017 – SeptoMatrix Cervical – 50 assorted matrices

0105040018 – SeptoMatrix Cervical – extra large matrices

0105040019 – SeptoMatrix Cervical – 50 large matrices

0105040020 – SeptoMatrix Cervical – 50 medium matrices

0105040021 – SeptoMatrix Cervical – 50 small matrices

0105040022 – SeptoMatrix Cervical – 50 extra small matrices

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Composition: Instruments: Polysulfone, Matrices: Polypropylène.

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