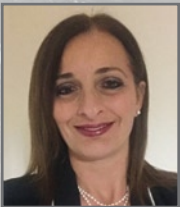
BioRoot™ RCS

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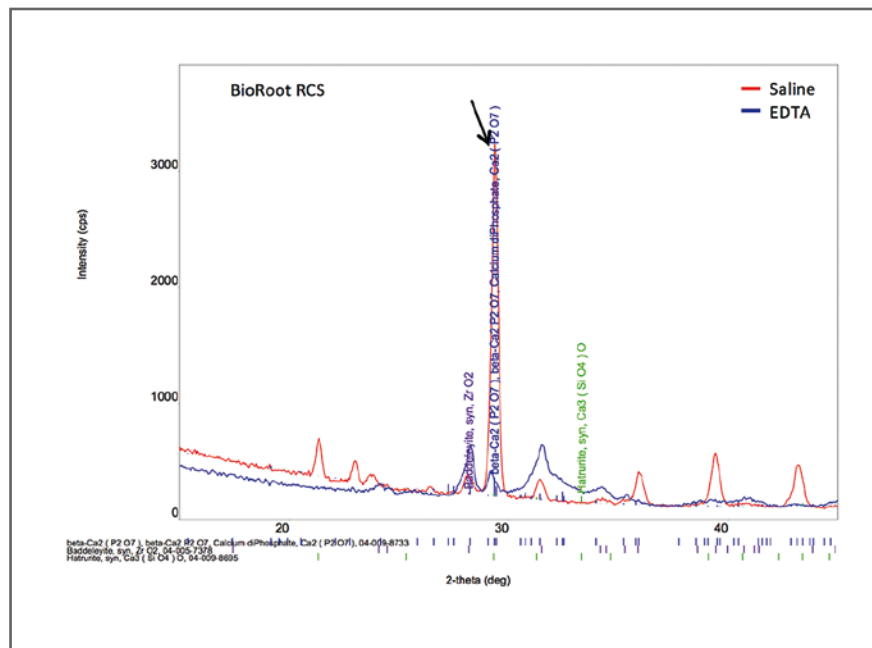
B.Ch.D., M.Phil., Ph.D., FICD, FADM, FIMMM, FHEA (UK)

School of Dentistry,

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College of Medical and Dental Sciences

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1. Holland R, de Souza V, Nery MJ, Otoboni Filho JA, Bernabé PF, Dezan Júnior E. Reaction of dogs' teeth to root canal filling with mineral trioxide aggregate or a glass ionomer sealer. *J Endod.* 1999 Nov;25(11):728-30.
2. Holland R, de Souza V, Nery MJ, Bernabé oF, Filho JA, Junior ED, Murata SS. Calcium salts deposition in rat connective tissue after the implantation of calcium hydroxide-containing sealers. *J Endod.* 2002 Mar;28(3):173-6.
3. Vizgirda PJ, Liewehr FR, Patton WR, McPherson JC, Buxton TB. A comparison of laterally condensed guttapercha, thermoplasticized gutta-percha, and mineral trioxide aggregate as root canal filling materials. *J Endod.* 2004 Feb;30(2):103-6.
4. Camilleri J, Montesin FE, Brady K, Sweeney R, Curtis RV, Ford TR. The constitution of mineral trioxide aggregate. *Dent Mater.* 2005 Apr;21(4):297-303.
5. Camilleri J. Hydration mechanisms of mineral trioxide aggregate. *Int Endod J.* 2007 Jun;40(6): 462-70.
6. Camilleri J. Characterization of hydration products of mineral trioxide aggregate. *Int Endod J.* 2008 May;41(5):408-17.
7. Monteiro Bramante C, Demarchi AC, de Moraes IG, Bernadineli N, Garcia RB, Spångberg LS, Duarte MA. Presence of arsenic in different types of MTA and white and gray Portland cement. *Oral Surg Oral Med Oral Pathol Oral Radiol Endod.* 2008 Dec;106(6):909-13.
8. Weller RN, Tay KC, Garrett LV, Mai S, Primus CM, Gutmann JL, Pashley DH, Tay FR. Microscopic appearance and apical seal of root canals filled with gutta-percha and ProRoot Endo Sealer after immersion in a phosphate-containing fluid. *Int Endod J.* 2008 Nov;41(11): 977-86.
9. Camilleri J, Montesin FE, Di Silvio L, Pitt Ford TR. The chemical constitution and biocompatibility of accelerated Portland cement for endodontic use. *Int Endod J.* 2005 Nov; 38(11):834-42.
10. Camilleri J. Evaluation of selected properties of mineral trioxide aggregate sealer cement. *J Endod.* 2009 Oct;35(10):1412-7.
11. Camilleri J, Mallia B. Evaluation of the dimensional changes of mineral trioxide aggregate sealer. *Int Endod J.* 2011 May;44(5):416-24.
12. Khalil I, Naaman A, Camilleri J. Properties of Tricalcium Silicate Sealers. *J Endod.* 2016 Oct; 42(10):1529-35.
13. Reszka P, Nowicka A, Lipski M, Dura W, Drożdżik A, Woźniak K. A Comparative Chemical Study of Calcium Silicate-Containing and Epoxy Resin-Based Root Canal Sealers. *Biomed Res Int.* 2016;2016:9808432.
14. Xuereb M, Vella P, Damidot D, Sammut CV, Camilleri J. In situ assessment of the setting of tricalcium silicate-based sealers using a dentin pressure model. *J Endod.* 2015 Jan;41(1): 111-24.
15. Prüllage RK, Urban K, Schäfer E, Dammaschke T. Material Properties of a Tricalcium Silicate- containing, a Mineral Trioxide Aggregate-containing, and an Epoxy Resin-based Root Canal Sealer. *J Endod.* 2016 Dec;42(12):1784-1788.
16. Camilleri J. Sealers and warm gutta-percha obturation techniques. *J Endod.* 2015 Jan;41(1): 72-8.
17. International Standard Organisation (2012). ISO 6876; Dentistry -- Root canal sealing materials.
18. Sarkar NK, Caicedo R, Ritwik P, Moiseyeva R, Kawashima I. Physicochemical basis of the biologic properties of mineral trioxide aggregate. *J Endod.* 2005 Feb;31(2):97-100.
19. Tay FR, Pashley DH, Rueggeberg FA, Loushine RJ, Weller RN. Calcium phosphate phase transformation produced by the interaction of the portland cement component of white mineral trioxide aggregate with a phosphate-containing fluid. *J Endod.* 2007 Nov;33(11): 1347-51.
20. Reyes-Carmona JF, Felipe MS, Felipe WT. Biomineralization ability and interaction of mineral trioxide aggregate and white portland cement with dentin in a phosphate-containing fluid. *J Endod.* 2009 May;35(5):731-6.
21. Atmeh AR, Chong EZ, Richard G, Festy F, Watson TF. Dentin-cement interfacial interaction: calcium silicates and polyalkenoates. *J Dent Res.* 2012 May;91(5):454-9.

22. Viapiana R, Moizadeh AT, Camilleri L, Wesselink PR, Tanomaru Filho M, Camilleri J. Porosity and sealing ability of root fillings with gutta-percha and BioRoot™ RCS or AH Plus sealers. Evaluation by three ex vivo methods. *Int Endod J*. 2016 Aug;49(8):774-82.
23. Reyes-Carmona JF, Felipe MS, Felipe WT. A phosphate-buffered saline intracanal dressing improves the biomineralization ability of mineral trioxide aggregate apical plugs. *J Endod*. 2010 Oct;36(10):1648-52.
24. Reyes-Carmona JF, Felipe MS, Felipe WT. The biomineralization ability of mineral trioxide aggregate and Portland cement on dentin enhances the push-out strength. *J Endod*. 2010 Feb;36(2):286-91.
25. Arias-Moliz MT, Camilleri J. The effect of the final irrigant on the antimicrobial activity of root canal sealers. *J Dent*. 2016 Sep;52:30-6.
26. Collado-González M, García-Bernal D, Oñate-Sánchez RE, Ortolani-Seltenerich PS(3), Lozano A, Forner L, Elena C, Rodríguez-Lozano FJ. Biocompatibility of three new calcium silicate- based endodontic sealers on human periodontal ligament stem cells. *Int Endod J*. 2016 Sep 26. doi: 10.1111/iej.12703. [Epub ahead of print]
27. Poggio C, Riva P, Chiesa M, Colombo M, Pietrocola G. Comparative cytotoxicity evaluation of eight root canal sealers. *J Clin Exp Dent*. 2017 Apr 1;9(4):e574-e578.
28. Eldeniz AU, Shehata M, Högg C, Reichl FX. DNA double-strand breaks caused by new and contemporary endodontic sealers. *Int Endod J*. 2016 Dec;49(12):1141-1151.
29. Dimitrova-Nakov S, Uzunoglu E, Ardila-Osorio H, Baudry A, Richard G, Kellermann O, Goldberg M. In vitro bioactivity of Bioroot™ RCS, via A4 mouse pulpal stem cells. *Dent Mater*. 2015 Nov;31(11):1290-7.
30. Camps J, Jeanneau C, El Ayachi I, Laurent P, About I. Bioactivity of a Calcium Silicate-based Endodontic Cement (BioRoot™ RCS): Interactions with Human Periodontal Ligament Cells In Vitro. *J Endod*. 2015 Sep;41(9):1469-73.
31. Harik R, Salameh Z, Habchi R, Camilleri J. The effect of irrigation with EDTA on calcium-based root canal sealers: a SEM-EDS and XRD study. *Journal of the Lebanese Dental Association* 2016; 49:12-23.
32. Donnermeyer D, Bunne C, Schäfer E, Dammaschke T. Retreatability of three calcium silicate- containing sealers and one epoxy resin-based root canal sealer with four different root canal instruments. *Clin Oral Investig*. 2017 Jun 22. doi: 10.1007/ s00784-017-2156-5. [Epub ahead of print].

Septodont Polska Sp. z o.o.

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tel.: +48 22 545 89 50

www.septodont.com.pl